


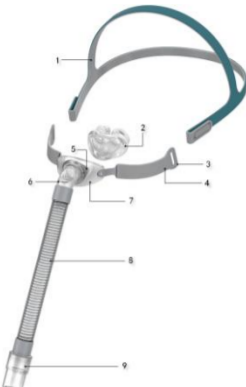
## INTERFACE COMPONENTS



- |            |  |
|------------|--|
| 1 Headgear | 2 Nasal Pillows (available in S, M, L) |
| 3 Panel    | 4 Soft sleeves                         |
| 5 Clip     | 6 Elbow                                |
| 7 Frame    | 8 Short tube                           |
| 9 Swivel   |  |

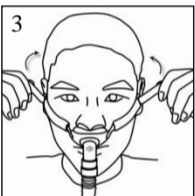
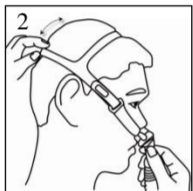
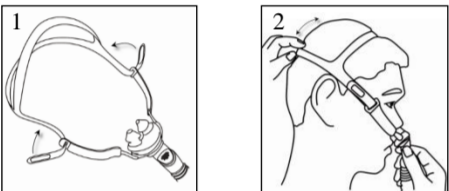
## Fitting

## Disassembly
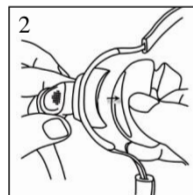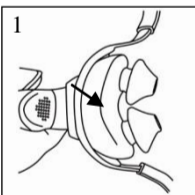
## Reassembly
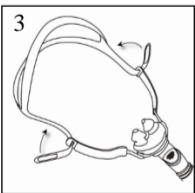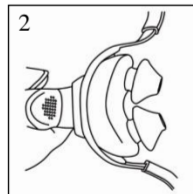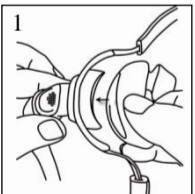
## P2 Nasal Pillows Interface (P2)

Thank you for choosing BMC's P2 Nasal Pillows Interface (P2). The nasal pillows interface is light, flexible and quiet. It is designed to minimize contact with your face, thus ensuring that you feel comfortable during therapy. This user manual provides you with the information you need for the correct use of your interface.

The interface is not made with natural rubber latex.

## Intended Use

The P2 Nasal Pillows Interface (P2) channels airflow non-invasively to a patient from a positive airway pressure device such as a continuous positive airway pressure (CPAP) or bi-level system.

The P2 Nasal Pillows Interface (P2) is:

- To be used by adult patients (> 66 lb / 30 kg) for whom positive airway pressure therapy has been prescribed.
- Intended for single-patient re-use in the home / institutional environment.

**CAUTION:** In the US, Federal law restricts the sale of this device to sale by or on the order of a physician.

## Contraindications

This interface should be used with caution for patients with following conditions:

- Facial deformity.
- Injured facial skin which will contact with the interface during use.

## Before Use The Interface

## ⚠️ WARNINGS

- This interface should be used only with flow generators (CPAP or Bi-level) recommended by your physician or respiratory therapist. It should not be used unless the flow generator is turned on and operating properly. The vent holes should never be blocked.
- Explanation of the Warning:** Flow generators are intended to be used with special interfaces with connectors which have vent holes (vent holes are located on the elbow) or separate exhalation devices to allow continuous flow of air out of the interface. When the flow generator is turned on and functioning properly, new air from the flow generator will flush the exhaled air out through the attached exhalation port. However, when the flow generator is not operating (eg, power failure), not enough fresh air will be provided through the interface, and exhaled air may be re-breathed. In this situation, remove the P2 immediately.
- Not applicable for silicone-allergy user.
- The exhaust vent flow is lower at lower interface pressure (see section "Technical Specifications" for Pressure-Flow Curve details). If the interface pressure during expiration is too low, there may not be enough exhaust vent flow to completely flush the air delivery hose of exhaled air before the start of the next inspiration.
- Discontinue using the P2 if you frequently experience nasal irritation, and consult your physician.
- If oxygen is used with the flow generator, the oxygen flow must be turned off when the flow generator is not operating. Explanation of the Warning: When the flow generator is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the flow generator enclosure. Oxygen accumulated in the flow generator enclosure will create a risk of fire. This warning applies to most types of flow generators.
- Please refrain from smoking while oxygen is in use.
- Images shown here are indicative only. If there is inconsistency between the image and actual product, the actual product shall govern.

## Fitting

- Thread the two side straps of the headgear through the gaps in panels separately, then fold back over to secure.
- Put the headgear over your head by holding the back strap and stretching it around the back of your head. The top strap should sit comfortably on top of your head.
- Adjust each end of the side strap evenly until the interface sits comfortably just under the nose. Reattach the fastening tabs.
- Move the interface away from your face then place it back to get a comfortable fit. Ensure the nasal pillows fit securely into your nostrils with both pillows positioned upright.
- Finally connect the end of the interface tubing into your device air tubing. Your interface is now fitted and ready to use.

**Notes:** Refer to your flow generator manual for details on setting and operational information.

Three nasal pillows sizes (S, M, L) are available, hold the pillows on your nostrils ensuring that the pillows size (eg, M) correctly fits for you.

If you feel air leak, readjust the angle of the nasal pillows to get a better seal. If you are unable to resolve any leaks, you may need an alternative nasal pillows size.

## Removal

To remove the P2, pull the nasal pillows assembly away from your nose and lift the headgear off your head. You do not need to loosen the headgear straps.

## Disassembly

- Holding the short tube on the interface, squeeze and pull the nasal pillows away from the clip in the interface frame.
- Remove the headgear from the interface.

## Notes:

- The short tube assembly cannot be taken apart.
- Do not pull on the short tube or the flow generator tubing as they may get damaged.

## Reassembly

- Attach the nasal pillows to the clip in the interface frame. Make sure the Nasal Pillows triangular opening and the frame platform opening is to point down.
- The Nasal Pillows is in the correct position. Make sure the frame and Nasal Pillows will fit tightly together without any gaps.
- Thread the two side straps of the headgear through the gaps in panels separately, then fold back over to secure.

## Cleaning the P2 in the Home

## Notes:

- Wash your hands before cleaning the interface.
- Wash the headgear before first time use, as the dye may run. It is important to always handwash the headgear.

## Daily / After Each Use

- Disassemble the interface components according to the disassembly instructions.
- Thoroughly clean the separated interface components (excluding headgear), by gently rubbing in warm (approx. 30°C) water using mild liquid detergent for 10 min.
- Use a soft bristle brush to clean the vent.
- Rinse all components except the Left and Right Panel well with drinking quality water and allow to air dry out of direct sunlight.
- When all components are dry, reassemble according to the reassembly instructions.

## Weekly

Handwash the headgear and all components in warm, (approx. 30°C) and mild liquid detergent for up to 10 minutes. Rinse the components well and allow them to air dry out of direct sunlight.

## ⚠️ CAUTIONS

- Always follow cleaning instructions and do not use solutions, containing bleach, chlorine, alcohol, aromatics, moisturizers, antibacterial agents, or scented oils to clean any part of the interface or air tubing, otherwise may damage the interface.
- Exposing any part of the interface or tubing to direct sunlight or heat may cause deterioration.
- If any visible deterioration of a component is apparent (cracking, crazing, tears etc), the component should be discarded and replaced.

## Reprocessing the Interface between Patients

This interface should be reprocessed when used between patients. Cleaning and disinfection instructions are available from the BMC website, [www.bmc-medical.com](http://www.bmc-medical.com). If you do not have internet access, please contact your BMC representative.

## Storage

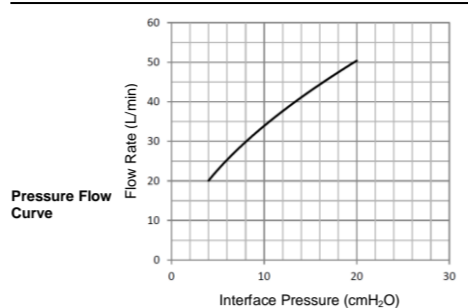
Ensure that the interface is thoroughly clean and before storing it for any length of time. Store the interface in a dry place out of direct sunlight.

Note: The interface does not contain PVC, DEHP or phthalates.

## Disposal

The P2 Nasal Pillows Interface (P2) does not contain any hazardous substances and may be disposed of with your normal house hold refuse.

## Technical Specifications

Pressure (cmH <sub>2</sub> O)	4	8	12	16	20
Flow Rate (L/min)	20	29	38	45	50

**Dead Space Information** Dead space is the empty volume of the nasal pillows and short tube assembly up to the swivel. The dead space of the P2 for Large pillows is 92 ml.

**Therapy Pressure** 4 to 20 cmH<sub>2</sub>O.

**Resistance** Drop in pressure measured (average for three nasal pillows sizes) at 50 L/min: 0.5 cmH<sub>2</sub>O; at 100 L/min: 1.6 cmH<sub>2</sub>O

**Sound** DECLARED DUAL-NUMBER NOISE EMISSION VALUES in accordance with ISO 4871. The A-weighted sound power level of the interface is 26 dBA, with uncertainty 3 dBA. The A-weighted sound pressure level of the interface at a distance of 1 m is 18 dBA, with uncertainty 3 dBA.

**Environmental Conditions** Operating temperature: +5°C to +40°C  
Operating humidity: 0% – 93% relative humidity non-condensing.  
Storage and transport: -20°C to +55°C  
Storage and transport humidity: 0% – 93% relative humidity non-condensing.

## Symbols

## Interface and Packaging

⚠️ Caution, consult accompanying documents. Indicates a Warning or Caution and alerts you to a possible injury or explains special measures for the safe and effective use of the device.

LOT Lot number

REF Catalog number

🔥 Temperature limitation

🌡️ Humidity limitation

🏭 Manufacturer

EC/REP Authorized Representative in the European Community

## Limited Warranty

It is warranted that P2 Nasal Pillows Interface (P2), including its components, frame, nasal pillows, headgear, tubing and other accessories shall be free from defects in material and workmanship for a period of ninety (90) days from the date of purchase by the initial consumer.

To exercise the rights under this warranty, contact the local authorized dealers. The expected service life of P2 Nasal Pillows Interface (P2) is one year.

**MANUFACTURER:**  
**BMC MEDICAL CO., LTD.**  
Room 110 Tower A Fengyu Building, No. 115 Fucheng Road, Haidian, 100036 Beijing, PEOPLE'S REPUBLIC OF CHINA

URL: [en.bmc-medical.com](http://en.bmc-medical.com)  
E-mail: [intl@bmc-medical.com](mailto:intl@bmc-medical.com)  
Tel: +86-10-51663880  
Fax: +86-10-51663880 Ext. 810

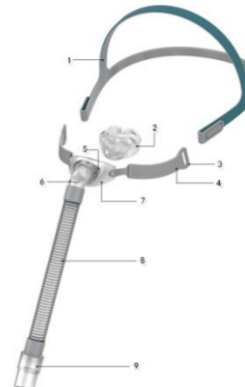
**EU AUTHORISED REPRESENTATIVE:**  
**Shanghai International Holding Corp. GmbH (Europe)**  
Eiffestraße 80, 20537 Hamburg, Germany

Tel: 0049-40-2513175  
Fax: 0049-40-255726

**Manufacturing Site:**  
**BMC (Tianjin) Medical Co., Ltd.**  
2/F North Area and 3/F, Building No.4, No.1 Xinxing Road, Wuqing District, (301700) Tianjin, P.R.China  
Tel: +86-22-82939881

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