## Mini-Wright™ PEAK FLOW METER

Your Mini-Wright Standard Peak Flow Meter will help you to monitor your asthma by measuring your peak flow. This leaflet explains how to use and care for your peak flow meter and how to record the readings. Peak flow measurements are an indication of how well air is moving through your lungs. Changes in your peak flow may require special treatment to establish what is happening in your airways. With careful use the Mini-Wright standard peak flow meter would be expected to last for two years (2,000 uses).

### Caution

- Do not obstruct the holes at the end of the peak flow meter.
- Please note that any coughing or spitting into the meter will adversely affect your readings and should be avoided.
- Fungal growths may occur if the instrument has not been cleaned for some time.
- Store in a clean dry place.

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**Contents:**
1 Standard Peak Flow Meter complete with detachable universal mouthpiece and Instructions with peak flow diary.

**Disposal Note:**
The Mini-Wright peak flow meter can be disposed of as normal household waste.

### Troubleshooting

Should you take your peak flow measurement and get an unusually low reading, you mean that your peak flow meter is broken or that the reading is inaccurate and your asthma is getting worse. Check to make sure that the meter is not broken by sliding the pointer along the slot, making sure that it moves freely.

If you feel comfortable that you are following the instructions correctly and that your meter is still broken, contact us at the address below for advice. No matter what your peak flow readings are, if you have symptoms such as chest tightness, shortness of breath, coughing or wheezing you should contact your healthcare professional for advice.

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### Care and Use

This peak flow meter does not require maintenance (other than cleaning) or calibration for the two years following its first use. At the end of two years, it should be replaced. When not in use this device should be kept in a clean, dry area away from any hazardous substances. It is important to keep your peak flow meter clean when in use. It should be cleaned at least once every six months and thoroughly if it has not been used for some time. The mouthpiece is detachable and should be kept clean using a proprietary disinfectant solution.

The Mini-Wright peak flow meter is suitable for multi-patient use. If Clement Clarke's validated cleaning instructions are followed. These are available on request, or via the internet at: www.clement-clarke.com

\* For Multi-Patient use see Care and Use.

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Rinse in clean warm water and shake gently to remove any excess water.

Allow it to dry thoroughly before using again.

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